



Lohmann & Rauscher

ReadyWrap® Full Range Brochure

Clinically effective¹ and easy-to-use^{2,3}
adjustable compression wrap



ReadyWrap

Adjustable compression garments for patients with venous and lymphatic diseases.

ReadyWrap®

Clinically-effective and easy-to-use.

ReadyWrap is an adjustable compression wrap that provides patients and their caregivers an easy-to-use solution to support wound healing and oedema management.¹

ReadyWrap provides clinically effective compression therapy while supporting patient compliance with its unique yet simple application method.



“To manage the swelling reduction, I placed him in bi-lateral ReadyWrap garments. He and his spouse loved that they could easily don/doff it on their own.”

Vicki Ralph, MPPA, OTR/L, CLT-LANA

Why choose ReadyWrap?

ReadyWrap features low-stretch straps that mimic short stretch bandaging — delivering effective compression therapy with high working and low resting pressure.

To support patient compliance, ReadyWrap offers color-coded VELCRO® straps for application and alternating strapping system to support one handed donning as needed.

With ReadyWrap, you can be certain your patients will have an effective and comfortable garment that easily adjusts to their limb and donning abilities.



Low-stretch material

Mimics short-stretch bandaging — the gold standard of treatment.



Color-coded straps

Provides easy step-by-step donning guidance.



50% Overlapping straps^s

Helps prevent gapping.¹



Blocked support spine

Helps maintain garment form to support all-day wear.¹




Foam Padding

Cushioning at elbow and wrist provides comfort and evenly distributes pressure.



Modular design

Garments work effectively on their own or in combination.

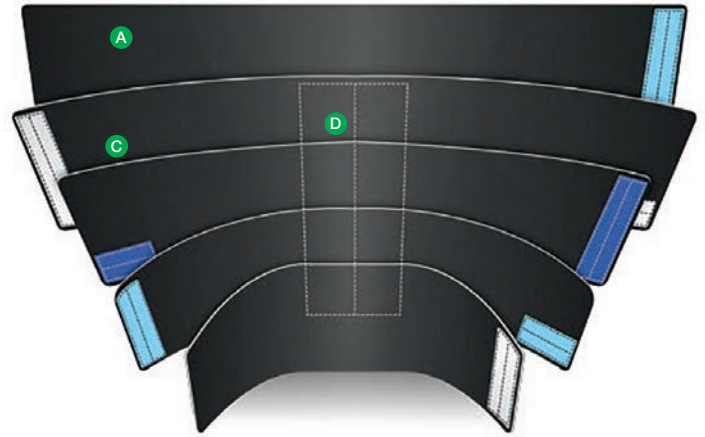

ReadyWrap Calf



ReadyWrap Arm

Indications

- Lymphoedema
- Chronic oedema
- Venous disease

Available in:



A full holistic and vascular assessment should be carried out before treatment.

ReadyWrap®

Adjustable compression system for upper limbs


Measuring Instructions

Circumference points:

- cA** around the metacarpal bones, around the base of the fingers
- cB** widest width of the palm, below the metacarpal phalangeal joints
- cC** narrowest point of the wrist
- cE** round the elbow joint and into the crease of the elbow
- cF** widest part of the bicep
- cG** top of the arm, just below the axillary crease

Lengths:

- IA-C** the length between point cA and point cC
- IC-G** the length between point cC and point cG

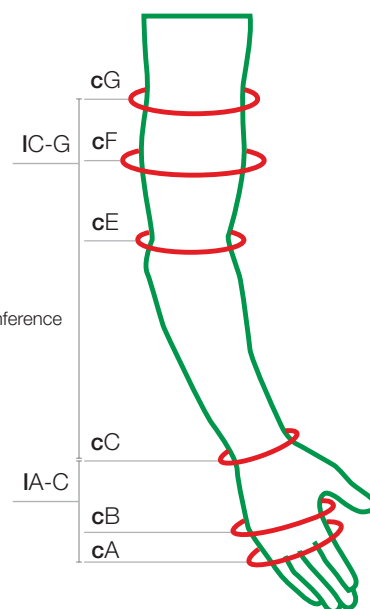

ReadyWrap Arm

- Double-sewn liner allows for easy, one-handed donning and doffing
- Padded wrist and elbow to eliminate compression hot spots and provide additional comfort
- Bilateral design contours to both left and right arms

	Circumference (cm)			Length (cm) (IC-G)	REF	
	Wrist (cC)	Elbow (cE)	Axilla (cG)		Beige	Black
Small	13.5-17	19-26	22-29	Short (up to 43cm)	104514	104515
				Average (up to 46cm)	104516	104517
				Tall (up to 49.5cm)	104518	104519
Medium	15.5-20	24-33	26-36	Short (up to 43cm)	104508	104509
				Average (up to 46cm)	104510	104511
				Tall (up to 49.5cm)	104512	104513
Large	17.5-23	29-40	31-43	Short (up to 43cm)	104502	104503
				Average (up to 46cm)	104504	104505
				Tall (up to 49.5cm)	104506	104507
X-Large	17.5-23	30-43	36-50	Short (up to 43cm)	104520	104521
				Average (up to 46cm)	104522	104523
				Tall (up to 49.5cm)	104524	104525



ReadyWrap Gauntlet

- Left and right specific design contours more naturally to the hand
- Built-in padding for palm and dorsum helps provide consistent compression over bony prominences
- Pocket design allows for foam pads to provide additional support
- Designed to overlap the ReadyWrap Arm garment to reduce gapping

	Circumference (cm)		Length (cm) (IA-C)		REF	
	Palm (cA)	Wrist (cC)			Beige	Black
Small	18-20	13.5-17.0	up to 12.5	Right	104498	104499
				Left	104490	104491
Medium	20-22	15.5-20.0	up to 13.5	Right	104496	104497
				Left	104488	104489
Large	22-24	17.5-23.0	up to 14.5	Right	104494	104495
				Left	104486	104487
X-Large	24-26	19.5-26.0	up to 15.5	Right	104500	104501
				Left	104492	104493



ReadyWrap®

Adjustable compression system for lower limbs


Measuring Instructions

Circumference points:

cA	around base of toes	cD	2-3 cm below popliteal fossa
cA¹	around the arch/dorsum of the foot	cE¹	5 cm above the knee
cB	2 cm above malleolus	cG	2 cm below gluteal fold
cC	widest part of calf		

Lengths:

IA-Y	heel to base of toes
IZ	heel to the tip of the longest toe (Fusion Liner only)
ID-B	the length between point cB and point cD
IG-E¹	the length between point cE ¹ (5 cm above the knee) and point cG
ID-Y	bottom of the heel to point cD (Fusion Liner only)

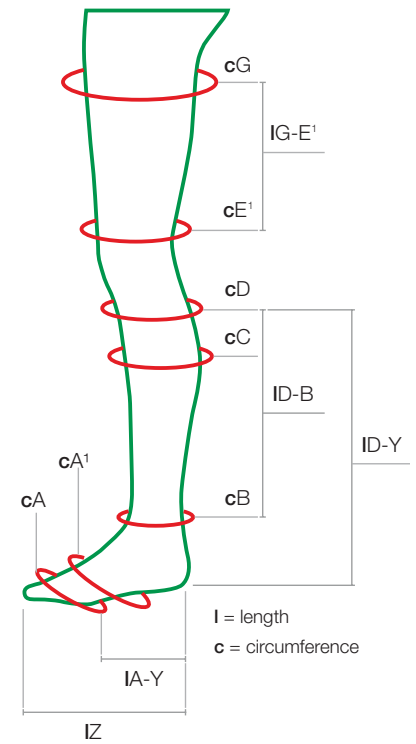

ReadyWrap Toes

- Unique design allows for adjustable toe girths
- Provides full foot coverage when paired with a ReadyWrap Foot unit
- Left and right specific design provides enhanced contouring

	Circumference (cm)		REF	
	Base of toes (cA)		Beige	Black
Small	18.0-20.4	Right	104416	104417
		Left	104406	104407
Medium	20.5-23.5	Right	104414	104415
		Left	104404	104405
Large	23.6-26.0	Right	104412	104413
		Left	104402	104403
X-Large	26.1-28.5	Right	104418	104419
		Left	104408	104409
XX-Large	28.6-31.0	Right	104420	104421
		Left	104410	104411



ReadyWrap Foot CT

- Colour coding and finger grips make it easy to apply
- Combining with ReadyWrap Calf is practical and easy
- Provides high containment for dorsum swelling
- Extensive range of sizes

	Circumference (cm)			Foot length (cm) (IA-Y)	REF		
	Base of toes (cA)	Foot (cA ¹)	Ankle (cB)		Beige	Black	
Small	18.0-20.4	20-25	18-23	16-19cm (Regular)	Right	141727	141729
					Left	141726	141728
				19-22cm (Long)	Right	141747	141749
					Left	141746	141748
Medium	20.5-23.5	24-29	20-28	16-19cm (Regular)	Right	141731	141733
					Left	141730	141732
				19-22cm (Long)	Right	141751	141753
					Left	141750	141752
Large	23.6-26.0	28-33	25-33	16-19cm (Regular)	Right	141735	141737
					Left	141734	141736
				19-22cm (Long)	Right	141755	141757
					Left	141754	141756
X-Large	26.1-28.5	32-37	30-38	16-19cm (Regular)	Right	141739	141741
					Left	141738	141740
				19-22cm (Long)	Right	141759	141761
					Left	141758	141760
XX-Large	28.6-31.0	36-41	35-43	16-19cm (Regular)	Right	141743	141745
					Left	141742	141744
				19-22cm (Long)	Right	141763	141765
					Left	141762	141764



ReadyWrap Foot SL

- Easy to fit in shoe with slim, tubular design
- Regular and Long lengths available to ensure coverage of the foot
- Left and right specific design to enhance contouring and improve fit
- Overlaps ReadyWrap Calf and ReadyWrap Toe units to help prevent gapping

	Circumference (cm)		Foot length (cm) (A-Y)		REF	
	Foot (cA ¹)	Ankle (cB)			Beige	Black
Small	21-24	18-23	16-19cm (Regular)	Right	144425	144427
				Left	144424	144426
			19-22cm (Long)	Right	144429	144431
				Left	144428	144430
Medium	25-28	20-28	16-19cm (Regular)	Right	144433	144435
				Left	144432	144434
			19-22cm (Long)	Right	144437	144439
				Left	144436	144438
Large	29-32	25-33	16-19cm (Regular)	Right	144441	144443
				Left	144440	144442
			19-22cm (Long)	Right	144445	144447
				Left	144444	144446
X-Large	33-36	30-38	16-19cm (Regular)	Right	144449	144451
				Left	144448	144450
			19-22cm (Long)	Right	144453	144455
				Left	144452	144454



ReadyWrap Fusion Liner (compressive)

- Offers compression at the foot and ankle
- Can be worn both day and night
- Compatible with ReadyWrap Calf
- Fits comfortably in your shoe

Size	Circumference (cm)			Length (cm)		REF	
	cA ¹	cB	cC	IZ	ID-Y		
Small	21-24	18-23	28-38	< 25	< 40	Regular	153523
				≥ 25	≥ 40	Long	153528
Medium	25-28	20-28	33-43	< 25	< 40	Regular	153524
				≥ 25	≥ 40	Long	153529
Large	29-32	25-33	38-48	< 25	< 40	Regular	153525
				≥ 25	≥ 40	Long	153530
X-Large	33-36	30-38	43-58	< 25	< 40	Regular	153526
				≥ 25	≥ 40	Long	153531
XX-Large	37-40	35-43	53-68	< 25	< 40	Regular	153527
				≥ 25	≥ 40	Long	153532



ReadyWrap Calf

- Can be worn with compression hosiery for additional support and protection
- Unique design reduces stretch and improves structural support
- Overlaps ReadyWrap Foot and ReadyWrap Knee garments to help prevent gapping

	Circumference (cm)		Length (cm) (ID-B)	REF	
	Ankle (cB)	Calf (cC)		Beige	Black
Small	18-23	28-38	up to 30	104450	104451
			up to 35	104452	104453
Medium	20-28	33-43	up to 30	104446	104447
			up to 35	104448	104449
Large	25-33	38-48	up to 30	104442	104443
			up to 35	104444	104445
X-Large	30-38	43-58	up to 30	104454	104455
			up to 35	104456	104457
XX-Large	35-43	53-68	up to 30	104784	104785
			up to 35	104786	104787



ReadyWrap Knee

- Easy, three-strap design provides a comfortable fit reducing pinching and high pressure points
- Overlaps ReadyWrap Calf and ReadyWrap Thigh to help prevent gapping

	Circumference (cm)		REF	
	Below Knee (cD)	Above Knee (cE')	Beige	Black
	Small	28-38	38-48	104464
Medium	33-43	45-58	104462	104463
Large	38-48	52-68	104460	104461
X-Large	43-58	59-78	104466	104467
XX-Large	53-68	66-88	104788	104789



ReadyWrap Thigh

- Low stretch material provides low resting and high working pressures from superior patella border to the groin
- Proximal strap helps accommodate straight and conical thighs
- Overlaps with the ReadyWrap Knee to help prevent gapping

	Circumference (cm)		Length (cm) (IE' - G)	REF	
	Above Knee (cE')	Groin (cG)		Beige	Black
	Small	38-48		48-63	up to 25
up to 30			104480		104481
Medium	45-58	58-73	up to 25	104474	104475
			up to 30	104476	104477
Large	52-68	68-83	up to 25	104470	104471
			up to 30	104472	104473
X-Large	59-78	78-93	up to 25	104482	104483
			up to 30	104484	104485
XX-Large	66-88	88-103	up to 25	104790	104791
			up to 30	104792	104793



ReadyWrap Extender Strap

- Provides an additional 10cm to a standard ReadyWrap strap
- "Y" clamp design helps ensure a secure fit
- Compatible with ReadyWrap Calf, ReadyWrap Knee and ReadyWrap Thigh

	REF	
	Beige	Black
One Size	304398	304399



ReadyWrap Liners (non-compressive)

Select the ReadyWrap Liner which is closest to the equivalent ReadyWrap garment

- Each ReadyWrap lower limb garment (excluding ReadyWrap Toes) is supplied with a ReadyWrap Liner
- Additional liners can be purchased in pairs
- Designed to be used under ReadyWrap garments to protect skin
- Provide no therapeutic compression

For ReadyWrap below Knee garments

	REF
	Black
Small - Large	149032
X-Large - XX-Large	149033

For ReadyWrap Thigh garments


	REF
	Black
Small - Large	149034
X-Large - XX-Large	149035



Contact your ReadyWrap distributor for Measurement and Order Forms

ReadyWrap® Lower Limb Measurement and Order Form

How to measure the leg:
Ask the patient to stand, if possible. Write your measurement results in the box adjacent to measuring point.

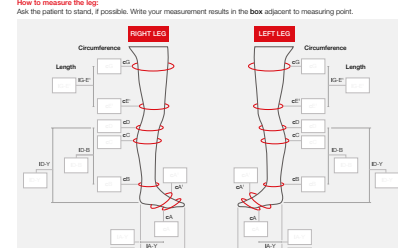

Circumference points:
 eA: around base of heel
 eA' : around the arch/bottom of the foot 2 cm above heel
 eB: widest part of calf
 eC: 2/3 cm below popliteal fossa
 eD: 2 cm above the knee
 eE: 2 cm below gluteal fold

Lengths:
 eA-C: heel to base of toes
 eA-D: heel to the tip of the longest toe (Pulsion Liner only)
 eA-E: the length between point eB and point eD
 eB-C: the length between point eC and 15 cm above the heel (Pulsion Liner only)
 eD-E: bottom of the heel to point eD (Pulsion Liner only)

NB: e = circumference, L = length
 All length measurements should follow body contour. All measurements to be in centimetres (cm).


Please mark (tick or cross) where applicable to indicate the garment choice and then enter the string, quantity and reference code.

Garment	Length	Foot	Colour	Size	Quantity	Code
Size			<input type="checkbox"/> Left <input type="checkbox"/> Right <input type="checkbox"/> Back			
Foot CT	<input type="checkbox"/> Regular <input type="checkbox"/> Large	<input type="checkbox"/> Left <input type="checkbox"/> Right <input type="checkbox"/> Back				
Foot BL	<input type="checkbox"/> Regular <input type="checkbox"/> Large	<input type="checkbox"/> Left <input type="checkbox"/> Right <input type="checkbox"/> Back				
Pulsion Liner	<input type="checkbox"/> Regular <input type="checkbox"/> Large	<input type="checkbox"/> Left <input type="checkbox"/> Right <input type="checkbox"/> Back				
Knee	<input type="checkbox"/> None <input type="checkbox"/> Back	<input type="checkbox"/> High <input type="checkbox"/> Back				
Thigh	<input type="checkbox"/> None <input type="checkbox"/> Back	<input type="checkbox"/> High <input type="checkbox"/> Back				
Summer Slip	<input type="checkbox"/> None <input type="checkbox"/> Back	<input type="checkbox"/> High <input type="checkbox"/> Back				
Liner	<input type="checkbox"/> None <input type="checkbox"/> High <input type="checkbox"/> Back	<input type="checkbox"/> High <input type="checkbox"/> Back				

www.Lohmann-Rauscher.com.au

ReadyWrap™ Upper Limb Measurement and Order Form

How to measure the arm:
Measure the arm in a supported and relaxed position with the elbow slightly flexed. The measurement points refer to the following anatomical landmarks:

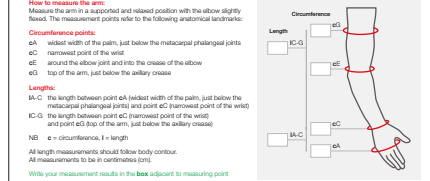

Circumference points:
 eA: widest width of the palm, just below the metacarpal (phalangeal) joints
 eB: narrowest point of the wrist
 eC: around the elbow joint and into the crease of the elbow
 eD: top of the arm, just below the axillary crease

Lengths:
 eA-C: the length between point eA (widest width of the palm, just below the metacarpal (phalangeal) joints) and point eC (narrowest point of the wrist)
 eA-D: the length between point eA (narrowest point of the wrist) and point eD (top of the arm, just below the axillary crease)
 NB: e = circumference, L = length
 All length measurements should follow body contour. All measurements to be in centimetres (cm).

Write your measurement results in the box adjacent to measuring point.

Order Information

Order Date: _____ Patient Name: _____
 Patient Phone: _____ Patient Email: _____
 Clinician Name: _____ Institution/Organisation: _____
 Clinician Phone: _____ Clinician Email: _____

Delivery

Attn: _____
 Address: _____
 City: _____ State: _____ Postcode: _____

Billing Information

Bill to: _____
 Account #: _____
 Card #: _____
 Exp: / / COV: _____

Products

ReadyWrap Arm (pair individually)	Style	Colour	Quantity
<input type="checkbox"/> Black <input type="checkbox"/> Beige	<input type="checkbox"/> Left <input type="checkbox"/> Right <input type="checkbox"/> Back <input type="checkbox"/> Beige		

Lohmann & Rauscher Pty Ltd, Suite 020, 21 Ferntree Gully Road, Mount Waverley, Victoria 3149, Australia
 Tel: +61 3 9578 8888 www.Lohmann-Rauscher.com
www.Lohmann-Rauscher.com.au

To discover more about ReadyWrap and additional resources, visit www.SupportforLife.com



For all queries relating to ReadyWrap as well as SwellSpots, Caresia, TributeNight or Exo products please contact your local distributor:

Sources:

- Ehmann, S; Bock, K. (2017). Critical evaluation of bandage alternative characteristics: a case series. EWMA Conference, Amsterdam, Netherlands.
- Hampton, S. (2016). The difficulty and the solution of compression therapy in a healed venous leg ulcer. Br J Community Nursing, 21(9), p34–p38.
- Ehmann, S. (2016). Multinational, pilot audit of a VELCRO adjustable compression wrap system for venous and lymphatic conditions. JWC, 25(9), p513–p520.



Scan here to view our extensive range of products in our latest Compression Catalogue.