

How Suprasorb® Liquacel Ag impresses your colleagues:



RESULTS



Excellent results:

100%
Atraumatic dressing change

100%
Removable in one piece

100%
Entrap cellular debris and remove with dressing change

99%
Conformability to the wound bed



Reduction of Biofilm

60%
complete

31%
partially



Achieving the treatment goal:

97%
Maintaining the moist wound environment

93%
Autolytic debridement

93%
Prevention of maceration

100%
Preventing from getting an infection

88%
Controlling the existing infection



Antimicrobial properties:

Reduction of inflammatory signs according to VAS scale:

-71%
Rubor (redness) Reduction

-75%
Calor (overheating) reduction

-63%
Functio laesa (restricted movement) reduction

-69%
Tumor (swelling) reduction

-64%
Dolor (pain) reduction

-76%
Wound odour reduction

DEMOGRAPHICS

Patients:
81

Average age:
67 Years

Wound depth:
59%


41%



superficial



deep

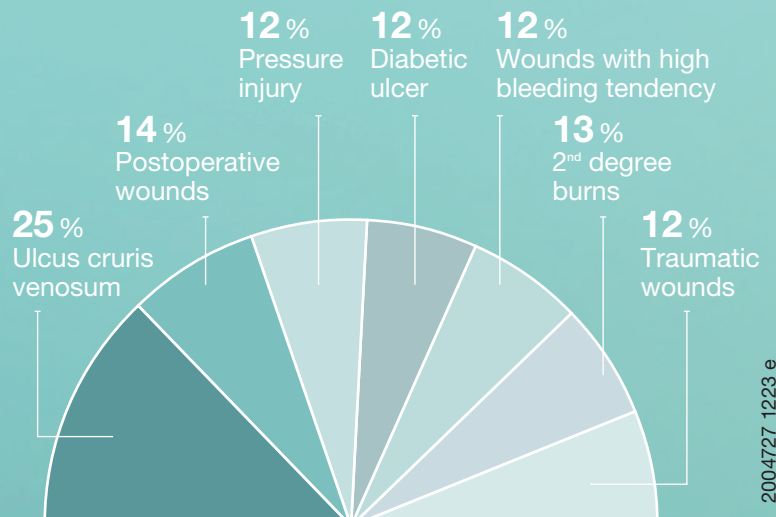

Source: Mete, E. et al. (2023) Observational, international, multicentre, single-arm study to assess the performance, safety and handling of new antimicrobial hydroactive fibre dressing [Poster], Wounds UK Conference 2023. 6 November.