



People.Health.Care.

Tribute® Wrap Bra **NEW**

Off-the-Shelf Foam Gradient Compression Garment



Solaris Collection™

Tribute® Wrap Bra

Where nightly comfort meets effective results.

Tribute® Wrap Bra is an off-the-shelf adjustable foam gradient compression garment for self-management during evening, nighttime, and low activity.



Off-the-Shelf



Evening & Nighttime



Low Activity



Proven foam technology

First-use chipped foam intended to create pressure differentials and promote a micro-massage-type effect to break up and soften hardened skin or fibrotic tissue to support skin health².



Powered by Coolcore®

Innovative interior fabric with chemical-free Cooling Power which rapidly wicks away sweat and moisture to regulate body temperature.



Easy-to-pull finger grip

Simple pull function to support self-management.



Adjustable strapping system

Easily conforms for the right, comfortable fit.



Channeling

Designed based on MLD principles of directing fluid to collateral pathways.



Tribute® Wrap Bra

Simply zip up & adjust!

Why choose Tribute® Wrap Bra?

Tribute® Wrap Bra supports easy self-care of lymphedema and edema in the upper torso during low activity by comfortably adjusting to contour your patients' chest and axillary region.

It helps maintain gains made during therapy, while allowing the flexibility to support further reduction. Simply zip it up and adjust the straps for the right fit in the comfort of your home!

Indications

- Various stages of lymphedema
- Edema



Black exterior
Slate interior

- 6-month warranty
- Easy garment care in washer and dryer



- A** Angled straps provide contour and shaping around lateral side of the chest
- B** Band adjustment strap accommodates a wide size range

- C** Shoulder straps stay aligned for easy positioning and comfort
- D** Crew neck provides contoured non-restrictive neckline
- E** Long front zipper pull makes it easy to put on and take off



People.Health.Care.

How to Order Tribute® Wrap Bra

1 Measure

Only 2 measurements needed.

2 Order


Available via your preferred garment supplier:

- Local DME
- National provider
- Online distributor

[US www.lohmann-rauscher.com/us-en/where-to-buy](http://www.lohmann-rauscher.com/us-en/where-to-buy)

[CA www.lohmann-rauscher.ca](http://www.lohmann-rauscher.ca)

Dealers can purchase direct from L&R via phone, fax, or online store.

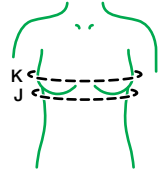

Tribute® Wrap Bra **NEW**

sold individually

Size	Circumference (cm)		REF
	J ^c	K ^c	
Small	77–92	85–105	163667
Medium	87–102	95–115	163668
Large	97–112	105–125	163669
X-Large	107–122	115–135	163670

Bra garment styles do not utilize or include a Sleep Sleeve.



Tribute Wrap Bra is part of a family of Tribute Wrap garments for patients with lymphedema or edema.

References

1. Bjork R, Ehmann S. S.T.R.I.D.E. Professional guide to compression garment selection for the lower extremity. Journal of Wound Care 2019; 28:(6 suppl 1):1–44
2. Chohan, A., Haworth, L., Sumner, S., Olivier, M., Birdsall, D., and Whitaker, J. (2019). Examination of the effects of a new compression garment on skin tissue oxygenation in healthy volunteers. Journal of Wound Care. 28. 1.