

Tribute® Wrap Bra

Adjustable foam gradient compression garment
Adaptives Kompressionssystem mit Schaumstoff und Druckgradient
Accessoire de compression médicale progressive en mousse réglable
Prenda de compresión graduada ajustable de espuma
Elemento a compresión graduada in schiuma regolabile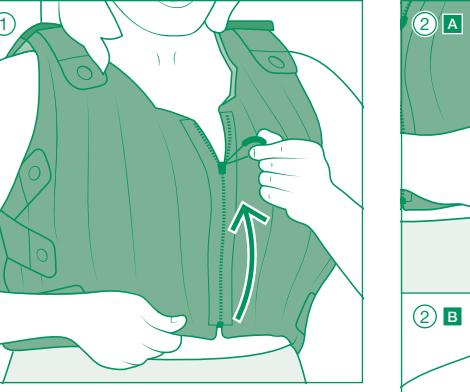

Verstelbaar schuimverband met compressiegradiënt
Adaptívny penový stupňovity kompresívny systém
Ayarlanabilir köpük degradé kompresyon giysisi
Justerbart gradient skumkompressionsystem
Justerbart system för gradientkompression i skummaterial
رداء إسنجي متدرج الشفط قبل التغليم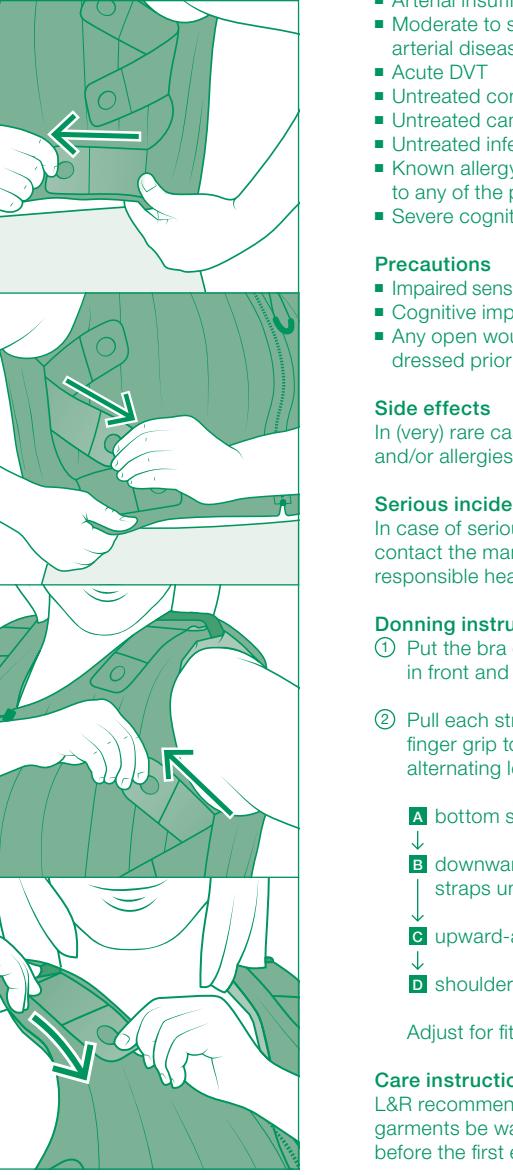

4052069/2968 / 2024-06

Symbol glossary provided at: <https://www.lohmann-rauscher.com/label-symbols/>

L&R USA INC.
5300 W Wheelhouse Road
Milwaukee, WI 53208 USA
www.Lohmann-Rauscher.com

EMERGO EUROPE
Vlaardingsdijk 60,
6827 AT Arnhem
The Netherlands

UK Responsible Person
Lohmann & Rauscher AG
1 Wellington Court - Lancaster
Park - Needwood - Burton
upon Trent - Staffordshire -
DE13 9PS - United Kingdom

en Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute Wrap Bra is an off-the-shelf adjustable foam gradient compression garment for the self-management of indicated conditions of the upper torso, particularly the breast and axillary region, during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue
- Help increase tissue oxygenation

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ① Put the bra on with zipper in front and zip closed.

- ② Pull each strap one at a time using the finger grip to contour to your shape, alternating left & right as follows:

- A bottom straps
- ↓ B downward-angled straps under bust
- ↓ C upward-angled straps above bust
- ↓ D shoulder straps

Adjust for fit and comfort.

Care instructions

L&R recommends all Tribute Wrap garments be washed and dried before the first extended use.

- A Fasten straps to garment prior to laundering.

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean
- Tumble dry low heat with other laundry items

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